


Rally™

Mini Polishers for Composite

These instructions are valid for all Rally Mini Polishers that are used for restorative and preventive work.

Single-use polishers are labeled with  and must not be used more than once, as multiple uses of these polishers can cause unnecessary risk to the patient and staff.

Composition: The polishers consist of synthetic/silicone rubber, pigments and abrasive. The shanks are made of medical plastic.

Indication: Rally Mini Polishers are used for polishing different materials which are specified composites & hybrid ceramics on the packaging label.

Contraindications: If a patient is known to be allergic to any of the above components, the polishers should not be used.

Adverse effect: None known to date.

General security advices:



Do not exceed maximum speed



Wear breathing protection



Wear eye protection



Use waterspray minimum 50ml/min

Precautions:

- Products may only be used in the specified field of application.
- Check the safe connection of the polisher before using it and perform a test run.
- Use proper hand piece only and remove any damaged, bent, or no longer central running polishers immediately.
- Avoid excessive pressure which can cause tilting, bending or blocking of the polisher.
- Insufficient water cooling can result in irreversible damage to the tooth and the surrounding tissue.
- Always keep track of LOT numbers of polishers to ensure traceability.

Storage: A dry, dust-free storage at 2 - 28°C (36 - 82°F) is recommended. Improper storage conditions can shorten the shelf life and cause malfunction of the product.

Expiry date: See imprint on label.

REMOVE



Choose the blue polishers for fast removal and contouring

SMOOTH



Choose the red polishers for smoothing and silky shine

SHINE



Choose the gray polishers for ultra high shine

USA Office

150 DeWitt Lane
Spring Lake, MI 49456
P 616.842.2244 F 616.842.2430

Europe Office

Carlstrasse 50
52531 Uebach-Palenberg, Germany
P +49.(0)2451.971.409 F +49.(0)2451.971.410

Keep out of reach of children. For use in dentistry only. Rx only. Caution: U.S. federal law restricts this device to be sold by or on the order of a licensed dentist.

For complete cleaning and sterilization instructions go to: garrisondental.com/sterilization

Garrison
Dental Solutions

Made in Liechtenstein

1739r1116 rev B